

USER MANUAL

OAKWORKS®

Boiance Shelf



Printed in U.S.A.

All rights are reserved. No part of this document may be photocopied, reproduced, or translated to another language without prior written consent of OAKWORKS[®], Inc.

OAKWORKS[®] is a registered trademark of OAKWORKS[®], Inc.

Notice

The information contained within this document is subject to change without notice and should not be construed as a commitment by OAKWORKS[®], Inc.

OAKWORKS[®], Inc. encourages requests for technical specifications and the like documentation to ensure accuracy. The appropriate documentation is available upon request.

OAKWORKS[®], Inc. shall not be liable for incidental or consequential damages in connection with or arising out of the furnishing, performance, or use of this document and the program material which it describes.

TABLE OF CONTENTS

PRODUCT USE DESCRIPTION / IMPORTANT SAFETY INSTRUCTIONS	4
PRODUCT USE DESCRIPTION	4
SYMBOL IDENTIFICATION.....	4
CONTRAINDICATIONS.....	4
IMPORTANT SAFETY INSTRUCTIONS	5
IMPORTANT SAFETY INSTRUCTIONS	6
BOIANCE SHELF ATTACHMENT INSTRUCTIONS.....	7
BOIANCE SHELF ATTACHMENT INSTRUCTIONS.....	8
BOIANCE SHELF ATTACHMENT INSTRUCTIONS.....	9
BOIANCE SHELF ATTACHMENT INSTRUCTIONS.....	10
BOIANCE SHELF ATTACHMENT INSTRUCTIONS.....	11
CLEANING & DISINFECTION	12
RECOMMENDED CLEANERS/DISINFECTANTS	12
CLEANING PROCESS	12
INSPECTIONS & WARRANTY	13
INSPECTIONS	13
WARRANTY	13
SPECIFICATIONS	13
PRODUCT SPECIFICATIONS	13
ENVIRONMENTAL CONDITIONS.....	13
CONTACT INFORMATION:.....	14

PRODUCT USE DESCRIPTION / IMPORTANT SAFETY INSTRUCTIONS

PRODUCT USE DESCRIPTION

The Boiance Shelf and bracket are used together to allow you to create the perfect place for client's arms when working on their back. The shelf can also be used alone as a head support for facials and other aesthetic treatments. The shelf is designed to allow therapists to relax their shoulders while working on clients, minimizing stress and injuries. It is intended to be used by a clinician in a spa or massage environment. No special training is required but a review of the following Safety Instructions is important for the safety of the operator and client.

SYMBOL IDENTIFICATION



This symbol, when used in this manual and on product labels, represents a caution warning. Be sure to read and comply with all precautions and warnings.



This symbol, when used in this manual and on product labels, indicates that the table and components are a Type B Applied Part pursuant to IEC 60601-1.



This symbol, when used in this manual and on product labels, indicates the name and address of the manufacturer.



This symbol, when used in this manual or on product labels, indicates the country of manufacture along with date of manufacture of the device next to it.



This symbol is used to indicate that the operator should consult the user manual.



Sitting is prohibited in this area.

CONTRAINDICATIONS

There are no known contraindications to the use of this equipment.

IMPORTANT SAFETY INSTRUCTIONS



CAUTION READ AND SAVE THESE INSTRUCTIONS

The Boiance Shelf is **not** designed to support the entire weight of the patient. Do not allow the patient to use in a manner to adjust themselves using their weight on the table or aid in the dismounting of the table.

The Boiance Shelf is designed to be a standalone device. It must not be modified or incorporated into any other equipment.

It is the responsibility of the operator of this equipment to ensure the proper operation of this accessory and it is especially important for the safety of the operator and client. Directions for use of this equipment are described in this manual. The operator should read these sections carefully.

Weight Limit: (patient's arms) 25 lbs. / 11.3 kg. Do not exceed.

Disposal of waste products, residues, etc., and accessories at the end of the expected service life are listed in the Oakworks Product User Manual listed on www.oakworks.com.

To reduce the risk of injury to persons:

1. Never operate this device if it is not working properly, or if it has been damaged. Contact OAKWORKS® Customer Service before use if damage is found.
2. Do not use it outdoors.

MISE EN GARDE LIRE ET CONSERVER CES INSTRUCTIONS

L'étagère Boiance n'est pas conçue pour supporter tout le poids du patient. Ne laissez pas le patient utiliser de manière à s'ajuster en utilisant son poids sur la table ou à aider à démonter la table.

L'étagère Boiance est conçue pour être un appareil autonome. Il ne doit pas être modifié ou incorporé dans un autre équipement.

Il est de la responsabilité de l'opérateur de cet équipement d'assurer le bon fonctionnement de cet accessoire et il est très important pour la sécurité de l'opérateur et du client. Le mode d'emploi de cet équipement est décrit dans ce manuel. L'opérateur doit lire attentivement ces sections.

Limite de poids : (bras du patient) 25 lb. / 11.3 kg. Ne pas dépasser.

L'élimination des déchets, des résidus, etc., et des accessoires à la fin de la durée de vie prévue est indiquée dans le manuel d'utilisation des produits Oakworks figurant sur www.oakworks.com.

Pour réduire le risque de blessure :


1. N'utilisez jamais cet appareil s'il ne fonctionne pas correctement ou s'il a été endommagé. Contactez le service client OAKWORKS® avant utilisation si des dommages sont constatés.

2. Ne l'utilisez pas à l'extérieur.

BOIANCE SHELF ATTACHMENT INSTRUCTIONS

Installation As an Arm Rest

- 1) Attach the Boiance Shelf bracket to the QuickLock face rest platform cross member.

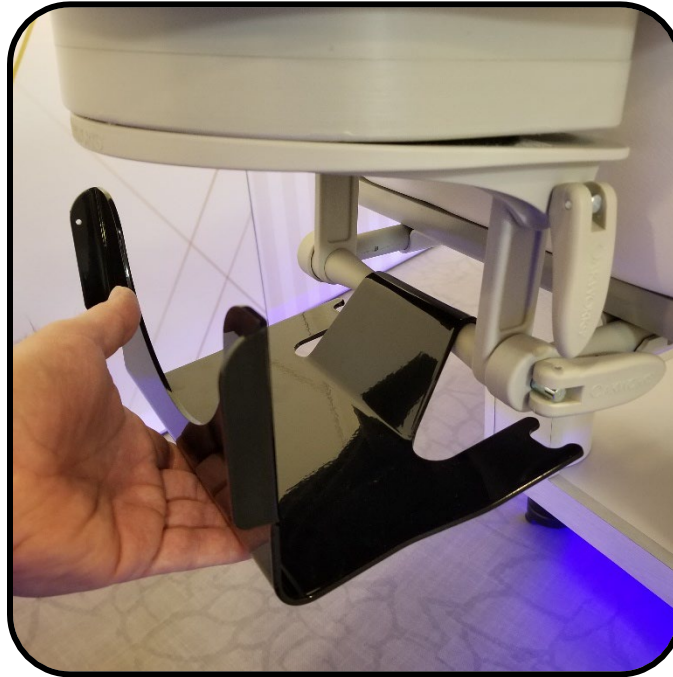

- 2) Pivot the bracket down until it rests on the QuickLock platform's dowels.



BOIANCE SHELF ATTACHMENT INSTRUCTIONS

- 3) Place the shelf on the bracket end by aligning the guides on the shelf to the bracket sides.



- 4) Hold the back of the bracket and push firmly on the shelf until it is fully seated on the bracket.



BOIANCE SHELF ATTACHMENT INSTRUCTIONS

- 5) The shelf guide and bracket are designed to have a tight fitting. Hold the bracket to help with removing and installing the shelf.




Installation As a Table Extender

- 1) Remove the face rest and riser from the QuickLock platform.



BOIANCE SHELF ATTACHMENT INSTRUCTIONS

- 2) Align the Boiance Shelf top slots to the QuickLock face rest base plate.

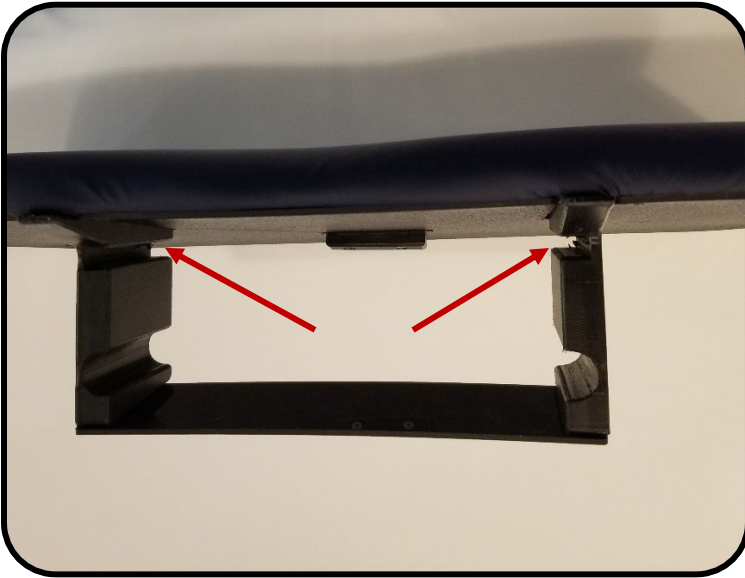

- 3) Push the Boiance Shelf on the QuickLock face rest base plate until it hits the hard stop block.



BOIANCE SHELF ATTACHMENT INSTRUCTIONS

- 4) Adjust the QuickLock platform cam locks to place the Boiance Shelf in the desired position.



CLEANING & DISINFECTION

RECOMMENDED CLEANERS/DISINFECTANTS

Reference the Recommended Cleaners and Disinfectant list (MMINML0008-EN) that came with the table. This information can also be found at www.oakworksmed.com under product information.

All cleaners and disinfectants could degrade the upholstery to some extent. However, following the recommended cleaner and disinfectant list and cleaning process will provide the best care for your table and support a long product life.

OAKWORKS® recommends a prepackaged wipe for cleaners/disinfectants to ensure best distribution of disinfectant for the required kill time, without leaving excess residue and/or overexposing components therefore minimizing the potential for damage to materials. Please read and follow disinfectants manufacturers' directions for cleaning and disinfection.

OAKWORKS® does NOT recommend the use of cleaners/disinfectants containing Hydrogen Peroxide, Acetic Acid, or Phenolics. These chemicals can cause damage to the appearance and/or material integrity of various components. Also, while the recommended cleaners/disinfectants list includes products containing Quaternary Ammonium compounds (“quats”), not all products containing quats are approved for use. Some contain additional detergents and/or surfactants which can damage some materials.

Use of non-approved cleaners or disinfectants may lead to damage to upholstery and other materials found on the table and will void the warranty.

CLEANING PROCESS

Follow the cleaners/disinfectant manufacturers' directions for use. Please note that cleaning and disinfecting OAKWORKS® products is a two-part process. First it must be cleaned of any visible soil, then it can be disinfected. Please follow this procedure for best results:

1. Using an approved cleaner or mild liquid soap and water, clean any visible soil off the Boiance Shelf. It is recommended that the upholstery be cleaned at least once a week to prevent disinfectant build-up.
2. Rinse with clean water and dry with a clean cloth or towel.
3. Using an approved disinfectant, thoroughly disinfect all surfaces of the Boiance Shelf, making sure they remain wet for the disinfectant manufacturer's recommended contact time. Do not allow disinfectant to pool on the upholstery after the recommended contact time.
4. Wipe off any excess liquid with a cloth or towel and clean water.
5. Dry all surfaces with a clean cloth or towel.

Avoid using writing instruments or other similar instruments around the upholstery as it can cause permanent staining. If this does occur, do not wipe with an alcohol based cleaner. Instead, blot the stain with a clean cloth/ paper towel. Use a recommended cleaner or disinfectant to remove the stain. Follow this with a rinse of clean water.

INSPECTIONS & WARRANTY

INSPECTIONS

RECOMMENDED REGULAR INSPECTIONS (monthly or local standard)

- Check for damage to the fabric, shelf, and bracket.
- Visually inspect for obvious damage that could cause problems during operation.

RECOMMENDED PERIODIC INSPECTIONS (yearly or local standard)

- Visually inspect the fabric, shelf, and bracket components for obvious damage that could cause problems during operation.
- Clean unusual buildup of dirt on the Boiance Shelf not normally cleaned on a regular basis.

WARRANTY

View complete warranty details at www.oakworksmid.com under the “INFO” tab, “Warranty and Return Policy.”

SPECIFICATIONS

PRODUCT SPECIFICATIONS

Weight	7.0 lbs. (0.2 kg.)
Shipping Weight	9.0 lb. (0.4 kg.)
Shipping Size	25"x16"x10"
Support Capacity	25 lbs. (11.3 kg.)

ENVIRONMENTAL CONDITIONS

CONDITIONS	TEMPERATURE	HUMIDITY	ATMOSPHERIC PRESSURE
Normal Use	50° (10°C) to 104° (40°C)	20% to 60% RH	98 to 105 kPa
Storage & Transport	-20° (-29°C) to 135° (57°C)	20% to 95% RH	98 to 105 kPa

USER MANUAL

OAKWORKS®

Boiance Shelf

CONTACT INFORMATION:



OAKWORKS® Inc.
923 East Wellspring Road
New Freedom, PA USA 17349

Phone (717) 235-6807

FAX (717) 235-6798

www.oakworksmed.com



EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



Original Instructions

Manual Part Number: Med-RA-PM-172

Revision level: 1

Revision Date: 03/20/2024

English, Printed in USA

www.oakworks.com · 717.235.6807



OAKWORKS® Spa